

“I Know My Status”



# HIV Home Test

Get your private results in only 20 minutes



**0800-123-4567**

Send a “please call me”

Lot 1206600 | Expires 05/2010

Manufactured in Thailand by ABC Company, PO Box 1234 Bangkok, tel 987-8765. Assembled in South Africa by XYZ Company, PO Box 2235 Durban, tel 0800 357 4125.



“I Know My Status”

Malik says: I know mine.


“I Know My Status”

Bongz says: I know mine.





# SET-UP

Before taking the Home HIV test.

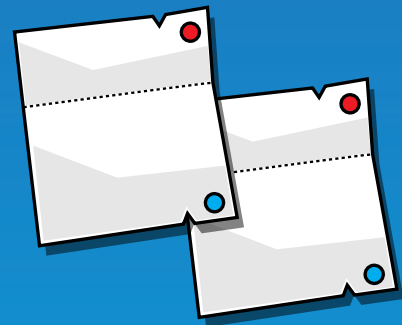

Questions?  
**0800 013 112**  
Send a "please call me"


1. Remove tubes from both packets.

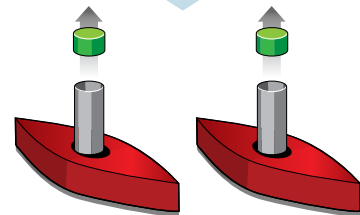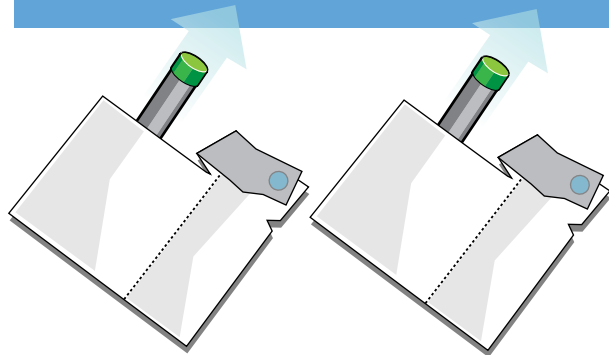


2. Remove top of each tube.  
Place tube in holder.  
Do not drink liquid.

# TEST


3. Remove swab from pouch

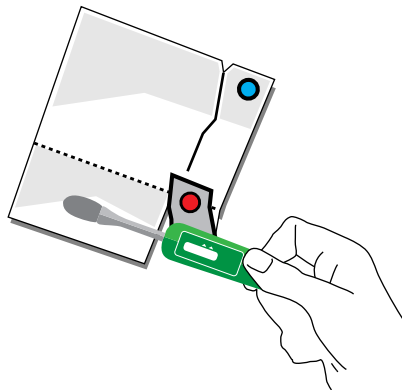


4. Swab top and bottom gums.

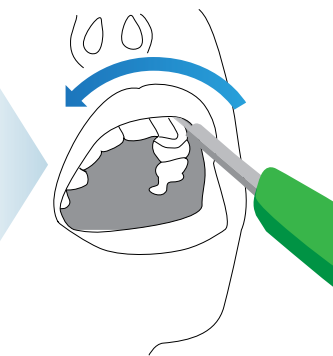


5. Place swab in the tube.  
Wait 20 Minutes

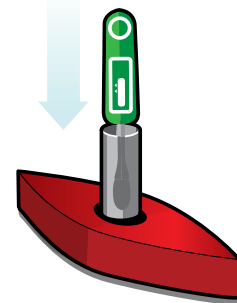




6. Remove swab, look for faint lines.

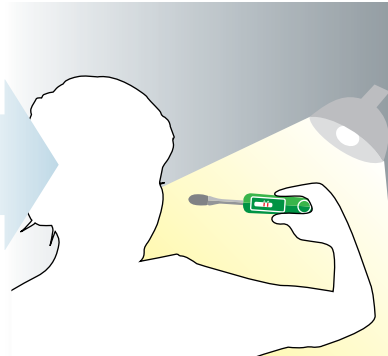

7. Repeat steps 3-5 with second swab.

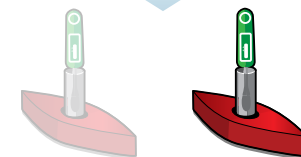

# RESULTS

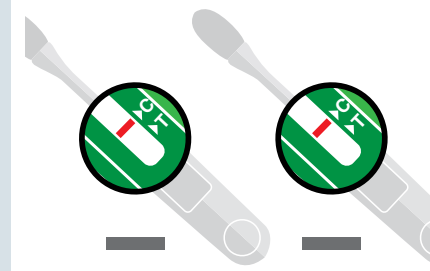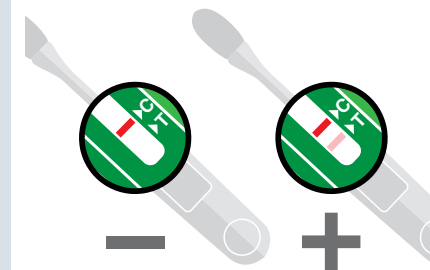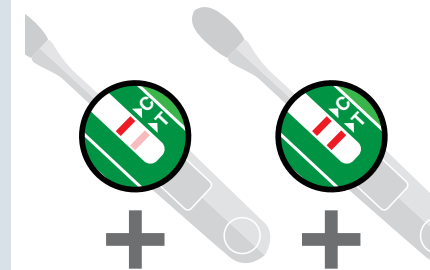

**Call now.**

 **0800 012 322**  
Send a "please call me"

Talk to a counselor,  
get help.

 **0800 013 334**  
Send a "please call me"

What you need to know.